
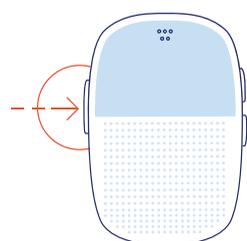


Getting Started


Meet Whisper, the new hearing system that gets better over time. Here are some helpful tips while you see if Whisper is right for you.

1. Charging the Whisper Brain. Please make sure to charge the Whisper Brain each night for at least 5 hours. To charge, insert the small end of the charging cable into the Brain, and the other end into the wall charger. Plug the charger into an outlet.

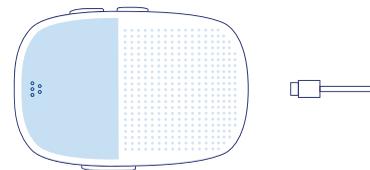


2. Turning the Whisper Brain on and off. The Whisper Brain can be turned on or off by pressing and holding the Settings button.

3. Turning on the earpieces. To turn the earpiece on, place a 675 hearing aid battery into the battery tray with its positive side visible. The earpiece will automatically turn on once the tray is closed.

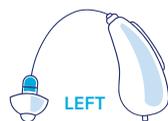


4. Putting the earpieces on. Please be sure the correct earpiece is in each ear. The right earpiece has a red tip, and the left, a blue tip. To put the hearing aids on, gently insert the domed tip at the end of the speaker wire into your ear canal, placing the remainder of the wire around the top of your ear so that the earpiece body sits comfortably behind your ear.

5. Adjusting the volume and settings. To adjust the volume, press the Volume buttons, and a short sound will play to confirm the level. To adjust the settings, press the Settings button to select between the Dynamic setting and the Custom setting.

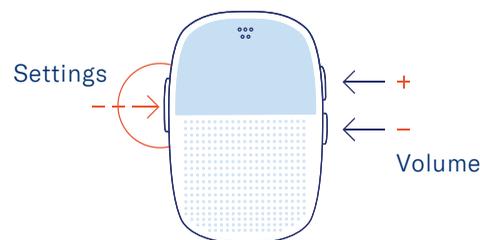

6. Connecting your phone to the Whisper Brain. Press the Settings and Volume Up buttons at the same time until the light on the Brain starts pulsing blue. Go to your phone's settings, and connect to the Whisper Brain as you would any other Bluetooth device. For specific instructions on how to do this on your phone model, please visit whisper.ai/devices.

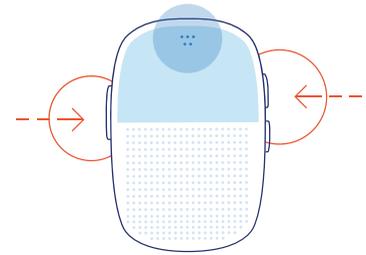

7. Using the Whisper mobile app for iPhone. Download the app at whisper.ai/iphone. The Brain must be turned on and within 10 feet of the phone in order to use the app. From the Whisper app, you can adjust your earpieces' volume (separately or together) and settings from your phone. Additionally, you can use the app to check the battery level of the Whisper Brain, and if the earpieces are connected to the Brain.

Have questions or need your Whisper Hearing System adjusted? Your hearing care professional is here to help.